

PIG Surgical Floor Skimmer

Unique shape and revolutionary design help keep your O.R. floors dry and safe!

Features / Benefits:

- **Shaped for efficiency** — cloverleaf shape directs liquids into the suction channel for maximum performance
- **High-absorbency wicking pad** — proprietary bottom wicking pad removes 60 liters per hour
- **Top fluid collection channel** — liquid running off the drapes can be easily collected from above and channeled to the wicking pad without ever hitting the floor
- **Low-profile design** — it's so thin, the surgeon will never know it's there
- **Easy-slide floor pad** — easily reposition or sweep across the floor to capture liquids no matter where they go
- **Silent suction** — helps keep the operating room quiet
- **12-foot suction tubing** — long tubing length allows access to all areas around the O.R. table (included with HC200)
- **Patent-pending design**

NOTE: PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.

PIG™ Surgical Floor Skimmer

| Item # | Description |
|--------|----------------------|
| HC200 | 10 Skimmers w/tubing |
| HC201 | 20 Skimmers |



9" x 9" Footprint



Bottom Wicking Pad




New Pig


Flip over for instructions >

HOW TO USE


PIG Surgical Floor Skimmer

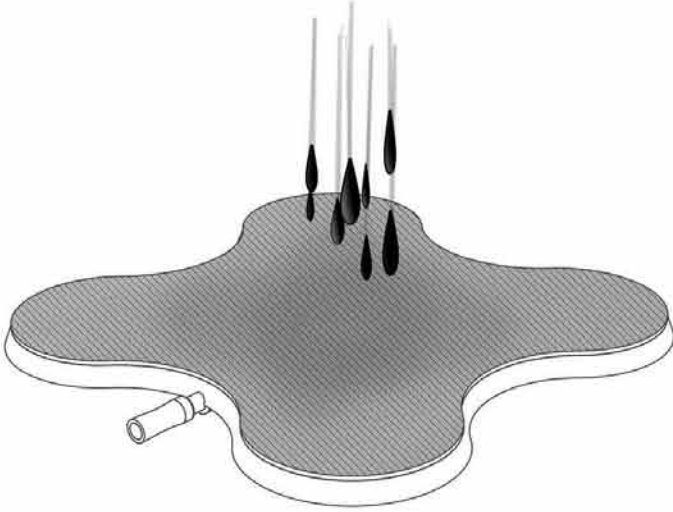

- 1.** Wet entire surface of bottom pad before use.

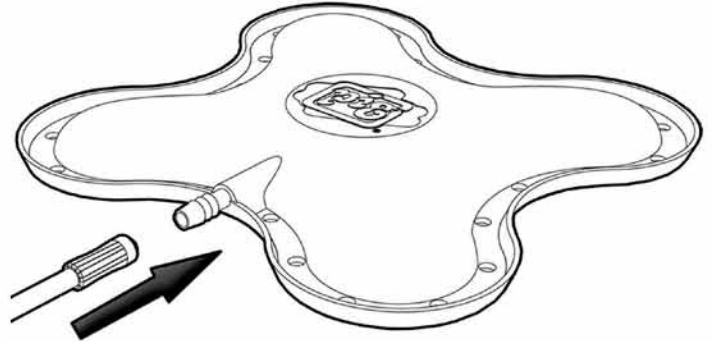

- 2.** Securely attach either end of suction tubing to barb fitting.

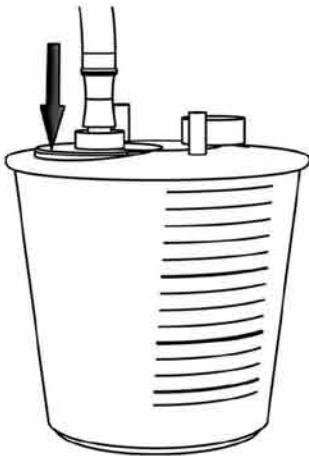

- 3.** Securely connect the other end of tubing to your fluid collection canister. Turn on your vacuum source when ready.



- 4.** Place skimmer wherever fluids are puddling — before, during or after a procedure.

NOTE: PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.



New Pig

Flip over for product details >

PIG™ Surgical Floor Skimmer

HC200 Tube Included, Latex-Free, 10 each

Revolutionary design attacks puddle problems from all sides to help keep O.R. floors dry and safe.

- Cloverleaf shape directs liquids into the suction channel for maximum efficiency and performance
- Proprietary bottom wicking pad removes 60 liters per hour
- Top fluid collection channel captures liquid from the drapes and channels it to the wicking pad
- Low-profile design is so thin, the surgeon will never know it's there
- Easy-slide floor pad can be repositioned or swept across the floor with your foot; capture liquids no matter where they go
- Silent suction design helps keep the operating room quiet
- Patent-pending design
- 12-foot suction tubing (included) allows access to all areas around the O.R. table



Specifications

| | |
|----------------|-------------------------------|
| Style | Tube Included |
| Color | Blue |
| Dimensions | 9" W x 9" L x 0.75" H |
| Configuration | Floor Skimmer |
| Latex | Latex-Free |
| Sold as | 10 per box |
| Weight | 8.6 lbs. |
| New Pig Patent | 9,451,854 |
| # per Pallet | 40 |
| Composition | Polypropylene/Cellulose Blend |
| UNSPSC | 42272014 |

Metric Equivalent

| | |
|------------|-------------------------------|
| Dimensions | 22.9cm W x 22.9cm L x 1.9cm H |
|------------|-------------------------------|

Weight

3.9 kg

Technical Information

Technical Documents

How to use the PIG Surgical Floor Skimmer



New Pig

World's best stuff for leaks, drips and spills.

One Pork Avenue • Tipton, PA 16684-0304

1-855-493-4647 • Fax: 1-800-621-7447 • newpig.com • hothogs@newpig.com